ultraMEDIC®

ultraMEDIC GmbH & Co. KG
Im Bruch 11-15
56567 Neuwied · Germany
Telefon:
+49(0) 26 31/9 69 83 -0
Telefax:
+49(0) 26 31/9 69 83 -33
email:
info@ultramedic.de

ultraMEDIC®

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ultraSTRAPS
Rückhalte-Gurtsystem
Restraining strap system
Art./Item SAN-0281

Instructions for Use

1.
Fix the cervical spine of the accident victim (with both the hands of the helper, if required, with the HWS rail).

2.
Position the S-BOON length-wise by the side of the patient.

3.
Two helpers should kneel on one side of the patient; one helper should get hold of one shoulder and the opposite thigh and the other helper, reaching partially across the first, should get hold of one hip and one knee of the patient.

4.
Upon the count of the helper by the head, the accident victim should be turned to his or her side, in axial alignment, as much as necessary to be able to push the spine board under the victim. As soon as the patient is lying on the spine board, the board, together with the patient, can be moved back into the horizontal position.
To do so, the helpers reach around in such a way that they can push the patient uniformly in the middle of the board.

5.
After the patient is positioned in the middle, the head can be fixed using the foam blocks.
(ultraHEAD-LOCK). The head must remain held firmly in the hands of the helper till both the fixing blocks are securely fastened.

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6a.

Fixing with the strap is done

1. at the height of the chest,
2. at the height of the pelvis and
3. below the patella.

If the patient is still not lying securely on the board, additional straps can be used, e.g. at the height of the ankle or thigh.

6b.

If the fixing on the S-BOON is done using the strap spider system (ultraSTRAP), the Y-shaped part of the system is positioned towards the shoulder, the rest of the strap is placed over the patient along the middle. The patient is strapped loosely from head to foot.

Thereafter, the Velcro straps are tightened in the same sequence till the patient is securely bound on the S-BOON.

7.

The patient is ideally transported with 3 to 6 helpers, who can each grip the board in the recesses provided at the side.