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1. Important information

1.1 Intended use

The head restraint system ultraHEAD-LOCK II is a head restraint system for use with spinal boards and intended for patients aged 2 years and older.

1.2 User groups

This product may only be operated by qualified, expert personnel.

2. Operating instructions

- Manual immobilisation may not be interrupted when applying the ultraHEAD-LOCK II on the patient.
- If the base plate or fixation frame is damaged, always use a new, faultless replacement part.

1. Place the patient onto the spinal board so that the head rests in the centre of the base plate. To ensure a neutral alignment, additional padding may be required under the head (in adults) or the thorax (in children).
2. Attach the base plate to the spinal board with the help of the broad restraint strap.
3. Open both restraint frame handles and guide the restraint frames into the rails up to the patient's head, then align the recesses with the ears.
4. To arrest, push both handles downwards. For possible adjustments, pull the handles upwards and adjust the restraint frames.
5. Attach both narrow restraint straps to the restraint frame. Centre the straps above the chin and forehead of the patient. When applying the ultraHEAD-LOCK II on a patient wearing a helmet (refer to local provisions), use adhesive tape instead of the restraint straps.
6. Before use, ensure that all straps are fastened tightly.

3. Technical Data

Order number:	SAN-0285	Delivery scope:
Designation:	ultraHEAD-LOCK II	1 Base plate
Weight:	0.6 kg	2 Restraint frames
Dimensions:	380 x 220 x 170 mm	3 Pads
Application:	-18°C to +43°C	2 narrow restraint straps
Storage:	-34°C to +52°C	1 broad restraint strap

4. Cleaning and preparation for reuse

1. Dismantling the equipment:

Remove both restraint frames from the base plate, and the base plate from the rescue board. Dispose of all restraint frame padding, head straps and base plate padding contaminated with body fluid.

The base plate and restraint frames can be reused after they have been appropriately cleaned.

2. Brushing or immersing in disinfectant:

Brush all parts with a warm water and dish-washing liquid solution for 4 minutes or immerse them in disinfectant (at 38°C with a concentration of 0.5%).

3. Rinsing and drying:

Rinse all parts with clean, warm (38°C) water and leave everything to air-dry completely at room temperature.

4. Attaching the padding:

If required, attach spare padding and straps.

5. Reattaching the base plate:

Reattach the base plate to the rescue board and store the restraint frames.

5. Limited warranty

ultraMEDIC grants the purchaser of this product a warranty period of 1 year following the date of purchase on any material and manufacturing errors. Any dispatch and processing costs are borne by the purchaser.

This limited warranty only applies if the product is stored, transported, used and maintained in accordance with the instructions contained in the operating instructions. ultraMEDIC cannot be held liable, neither within the scope of this limited warranty, nor in any other way for any damages, consequential damages, economic losses or material damages of any kind.

This limited warranty does not refer to normal wear and tear or to discolouration and other cosmetic irregularities, which do not pose an influence on the efficiency of the product.

Impressum:

ultraMEDIC GmbH & Co.KG

Im Bruch 11-15

D-56567 Neuwied

Tel. (02631) 96983-0

Fax (02631) 96983-33

info@ultramedic.de

www.ultramedic.de