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---

# Content

	<b>Page</b>
<b>1. Important information</b>	9
1.1 Liability and Warranty	9
1.2 User groups	9
<b>2. Information on the safety and protection of the product</b>	9
<b>3. Application</b>	9
3.1 Safety information	9
3.2 Application as scoop stretcher	10
3.3 Application as spine board	10
<b>4. Care and maintenance</b>	11
<b>5. Maintenance</b>	11
<b>6. Storage</b>	12
<b>7. Service life</b>	12
<b>8. Optional accessories</b>	12
<b>10. Identification and Warranty Certificate</b>	13

---

## 1. Important information

Before using the ultraCOMBI-STRETCHER, please read the operating manual and safety instructions carefully. Keep the operating manual in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH & Co. KG directly. Always specify the serial number to identify the ultraCOMBI-STRETCHER.

### 1.1 Liability and Warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts. Shipping or wage costs are not subject to warranty.

### 1.2 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the ultraCOMBI-STRETCHER.

## 2. Information on the safety and protection of the product

1. Before and after every use, check the components and patient restraining belts for damage and completeness.
2. Ensure that the patient restraining belts are correctly installed.
3. Before initial use, conduct a test run with an empty ultraCOMBI-STRETCHER.
4. In the event of damage or suspected damage to the straps or the ultraCOMBI-STRETCHER, remove the product from operation immediately until any occurring irregularities have been remedied.

## 3. Application

The ultraCOMBI-STRETCHER is a combined spine board and scoop stretcher used to immobilise injured persons who may have suffered a spinal fracture.

### 3.1. Safety information


1. A specialist medical assessment must be conducted before the injured party is moved, lifted or transported.
2. To ensure the patient's safety, at least 2 users are required to operate the ultraCOMBI-STRETCHER.
3. The injured party must be immobilised on the ultraCOMBI-STRETCHER during every use. Insufficient or no restraint may lead to severe patient or rescuer injuries.

---

## 3.2 Application as scoop stretcher

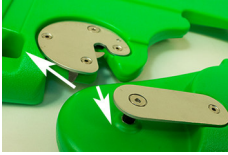**Figure 1**

1. To position the ultraCOMBI-STRETCHER underneath the patient without having to move him, open one of the metal fasteners. Unlock the snap buckle by activating the lock mechanism. (Figure 1)

***There are various application techniques for the scoop stretcher. You can place the halves under the patient from the sides or only open the scoop stretcher at the bottom end, which then forms an "V". Carefully position the top end under the patient's head and close the bottom end (with a so-called "scooping" motion).***

***The type of application must be individually determined, depending on the nature of the injury.***

2. If the patient's legs protrude beyond the edge of the ultraCOMBI-STRETCHER, check to see whether the patient's legs require additional immobilisation. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.

3. Once the parts have been locked (clearly audible clicking sound during the locking process), ensure that both snap locks of the ultraCOMBI-STRETCHER are locked and only then secure the patient with the supplied straps.

4. Secure all 3 patient safety straps as shown in Figure 2 . When locking the buckle, ensure that the mark on both sides of the buckle are on the same side. The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the straps.

5. Check all buckles and straps before lifting and transporting the patient.

## 3.3 Application as spine board

1. Before using the ultraCOMBI-STRETCHER as a spine board, ensure that both snap buckles are locked.


2. Similar to a spine board, the ultraCOMBI-STRETCHER is applied depending on the injury and local conditions.

3. If the patient's legs protrude beyond the edge of the ultraCOMBI-STRETCHER, check to see whether the patient's legs require additional immobilisation. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.

4. Secure all 3 patient safety straps as shown in Figure 2 . When locking the buckle, ensure that the mark on both sides of the buckle

are on the same side. The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the straps.

5. Check all buckles and straps before lifting and transporting the patient.

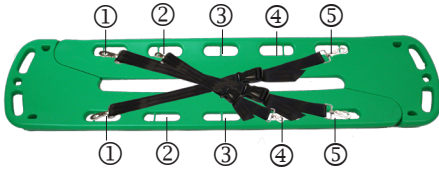**Figure 2**

*Diagonal fastening:*

*Strap 1 and 2: Recessed grips ① and ⑤*

*Strap 3: Recessed grips ② and ④*

## 4. Care and maintenance

Use a mild soap and warm water to clean the ultraCOMBI-STRETCHER and the straps.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleansing chemicals may damage the material and weaken the straps of the ultraCOMBI-STRETCHER.

The snap locks may not be dismantled for cleaning purposes. After cleaning the locks with a mild soap and warm water, they do not require additional lubrication.

## 5. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraCOMBI-STRETCHER and straps visually inspected by an expert on an annual basis.

For German users, we recommend consideration to the following regulations and guidelines:

MPBetreibV

DGUV Grundsatz 305-002

DGUV Regel 112-198/199

**The following parts must be checked and the following defects must be excluded:**

**Base plate:** deformation, tears, breaks, loosened screws, broken parts

**Karabiners:** impaired function, deformation, torsion, tears and breaks

**Straps:** Cutting and abrasion points, intact seams, tissue-damaging stains

**Metal parts:** Rust and oxidation

**other damages**

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

---

## 6. Storage

Store the ultraCOMBI-STRETCHER in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

We recommend storing the ultraCOMBI-STRETCHER as one unit in order to avoid the loss of individual parts and possible damages.

## 7. Service life

The service life of the ultraCOMBI-STRETCHER depends on the individual conditions of use. Depending on how frequently and intensely the rescue stretcher is used, wear and tear on the ultraCOMBI-STRETCHER is normal. In this regard, correct use is assumed.

The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition or **no later than after 12 years**.

The lifecycle of the optional accessories is described in the associated operating instructions.

## 8. Optional accessories

SAN-0284	ultraHEAD-LOCK I	Head restraining set
SAN-0285	ultraHEAD-LOCK II	Head restraining set
SAN-0286	ultraSTRAPS Xtreme	Safety strap system with clip locks
SAN-0281	ultraSTRAPS	Safety strap system with Velcro fastener
SAN-0280-T	ultraSPINE BAG	Storage bag
SAN-0084-S	ultraBELT	Safety strap set
SAN-0260-1	X-BOON ONE	Vacuum mattress
SAN-0278	ultraVACPUMP I	Vacuum pump

**Identifizierungs- und Gewährleistungszertifikat**  
**Identification and Warranty Certificate**  
**Certificat d'identification et de garantie**  
**Certificado de identificación y de garantía**

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