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## Description:

---

- 1) Unique head strap is designed to conform patient's forehead.
- 2) Large ear holes for monitoring the patient's bleeding
- 3) Minimal interference with X-ray, MRI, or CT procedures
- 4) Waterproof plastic coating makes it easy to prevent bacterial growth within components. Will not absorb blood or bodily fluids. Durable and easy to clean.
- 5) Direct contact with skin, can be used in very cold weather.

## Check list:

---

Each unit should have: 1 Base Board with 2 "D" shape loop and 2 velcro loops on each side, 2 Block with round hole, 2 Head Strap.

## How to use:

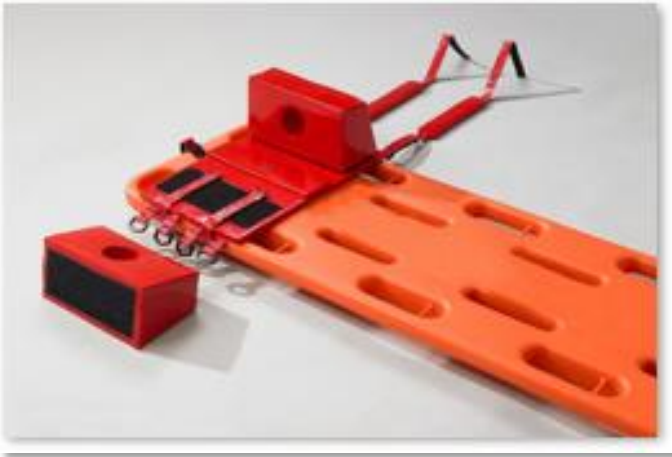
---

1) Use central top belt

Put base board on spine board, insert the top belt through the central hole on spine board.

Have belt go through the D loop, reverse and stick on belt velcro.

2) Use two belt to fix base board on spine board.  
Insert the two lateral fastening straps through the handles of the board to the rear and pull them on the back of the board through the opposing handles back up. Tighten the straps with Velcro under slight tension on the baseplate.

3) Use two Block to immobilize patient's head  
Put the right block with the straight side in head towards the patient's head and fix it to the Velcro level. Repeat this procedure now to the second block. Pay attention to a slight "pinch" of the head and on the correct fit in the ear region of the patient.

#### 4) Put on two Head Strap

Take one of the two belts at hand. Open one side of the Velcro strap and pull it through the D - ring of the base plate at height of forehead. Then close the Velcro. Now open the Velcro fastener on the other side of the belt and pull it on the opposite side also at forehead level. Pull the belt slightly tight and close the Velcro.

Repeat this procedure at the level of the chin.

## Maintenance:

---

Clean with wet cloth after use. Keep dry, clean and ventilated

**SmartEm GmbH**

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56427 Siershahn · Germany

Telefon:

+49(0) 26 23 / 86 99 29-0

Telefax:

+49(0) 26 23 / 86 99 29-9

Email: [info@smartem.de](mailto:info@smartem.de)

Web: [www.smartem.de](http://www.smartem.de)