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General

The ultraPULL STRAP SET consists of 2 length-adjustable pull straps and a carabiner. The product is used as a flexible support when pulling basket stretchers on steep slopes or rough terrain. The ultraPULL STRAP SET is approved for all ultraMEDIC basket stretchers. It can be pre-mounted on a basket stretcher without tools.

Important information

Please read the instructions for use and safety instructions carefully before you start using the ultraPULL STRAP SET. Keep the instructions for use in a safe place to avoid operating errors. Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly.

Warranty terms

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the product.

Information on the safety and protection of the product

1. Before and after every use, check that the carabiner and straps are intact and complete.
2. Make sure that ultraPULL STRAP SET is installed correctly. Use **ONLY** the attachment points (metal eyelets) on the head side of the basket stretcher.
3. The ultraPULL STRAP SET is approved for a maximum load capacity of 350 kg **ONLY** when using the two straps and for pulling in the longitudinal direction.
4. While using the pull straps, make sure that they are not pulled through sharp edges, sharp or rough objects and are not damaged as a result.
5. Before using it for the first time, do a test run with the ultraPULL STRAP SET to learn how to handle the product.
6. Observe the principles of use to avoid the risk of falling.
7. In the event of damage or suspected damage to the straps or carabiner, remove the product from operation immediately until any occurring irregularities have been remedied.

Cleaning and care

Use mild soapy water and warm water (40°C) to clean the ultraPULL STRAP SET. You can use commercially available alcohol-based agents for disinfection. Other solutions or cleaning chemicals can damage the material of the straps and carabineer.

Wet belts must only dry naturally and must be kept away from direct heat.

Maintenance

The ultraPULL STRAP SET is not repairable.

In order to ensure the safety of the injured party and the user, we recommend that you have the ultraPULL STRAP SET visually inspected by an expert at least once a year.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002

DGUV Regel 112-198/199

For users outside of Germany, regular testing in accordance with country-specific is essential.

The following parts must be tested to exclude the following defects:

Straps	Cutting and abrasion points, damage of the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Other damage	

In the event of damage or suspected damage, remove the product from operation immediately.

Storage

Store ultraPULL STRAP SET in a dry and dark place. Avoid direct sunlight and direct heat radiation from heating sources. These factors can damage the material.

Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use.

The service life of the ultraPULL STRAP SET depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals, aggressive atmosphere and dirt. The product should be phased-out depending on the condition or at the latest after 6 to 8 years.

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